


IntelViewer for MacOS

1. Download IntelViewer from this link:


[San Radiology & Nuclear Medicine: IntelePACS: IntelViewer Tracks](#)

2. Click on Stable Track Mac OS X Installer

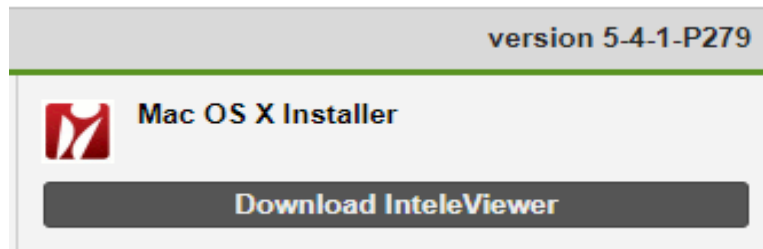

3. Download the Installer and install the appt


4. After the installation, this window will pop up. Drag IntelViewer to the applications folder

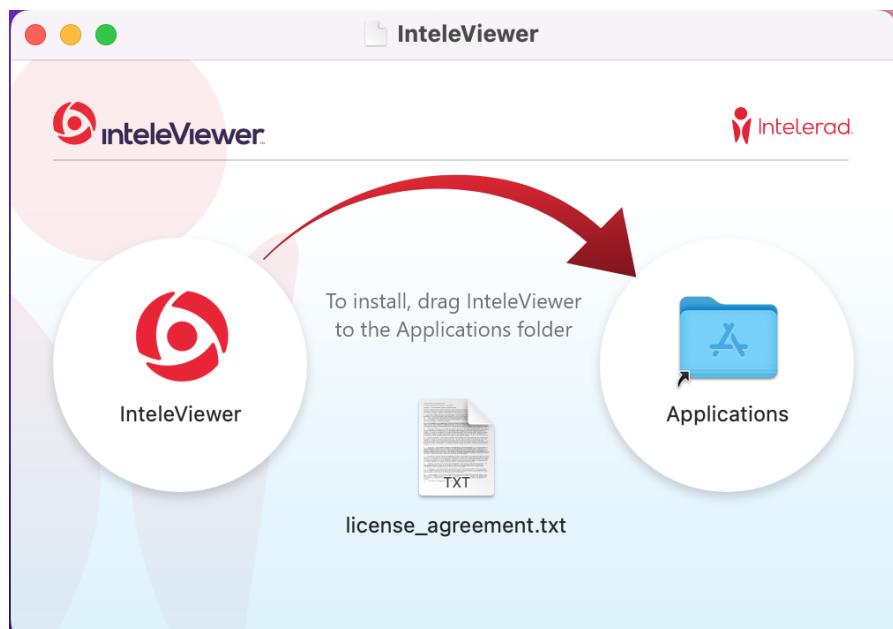

5. Launch IntelViewer from the Applications folder




6. You will get the below security prompt – click CANCEL:

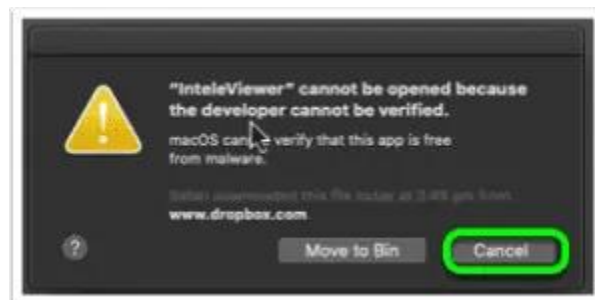


7. You will then need to go to System Preferences > Security & Privacy



8. Click on OPEN ANYWAY for IntelViewer

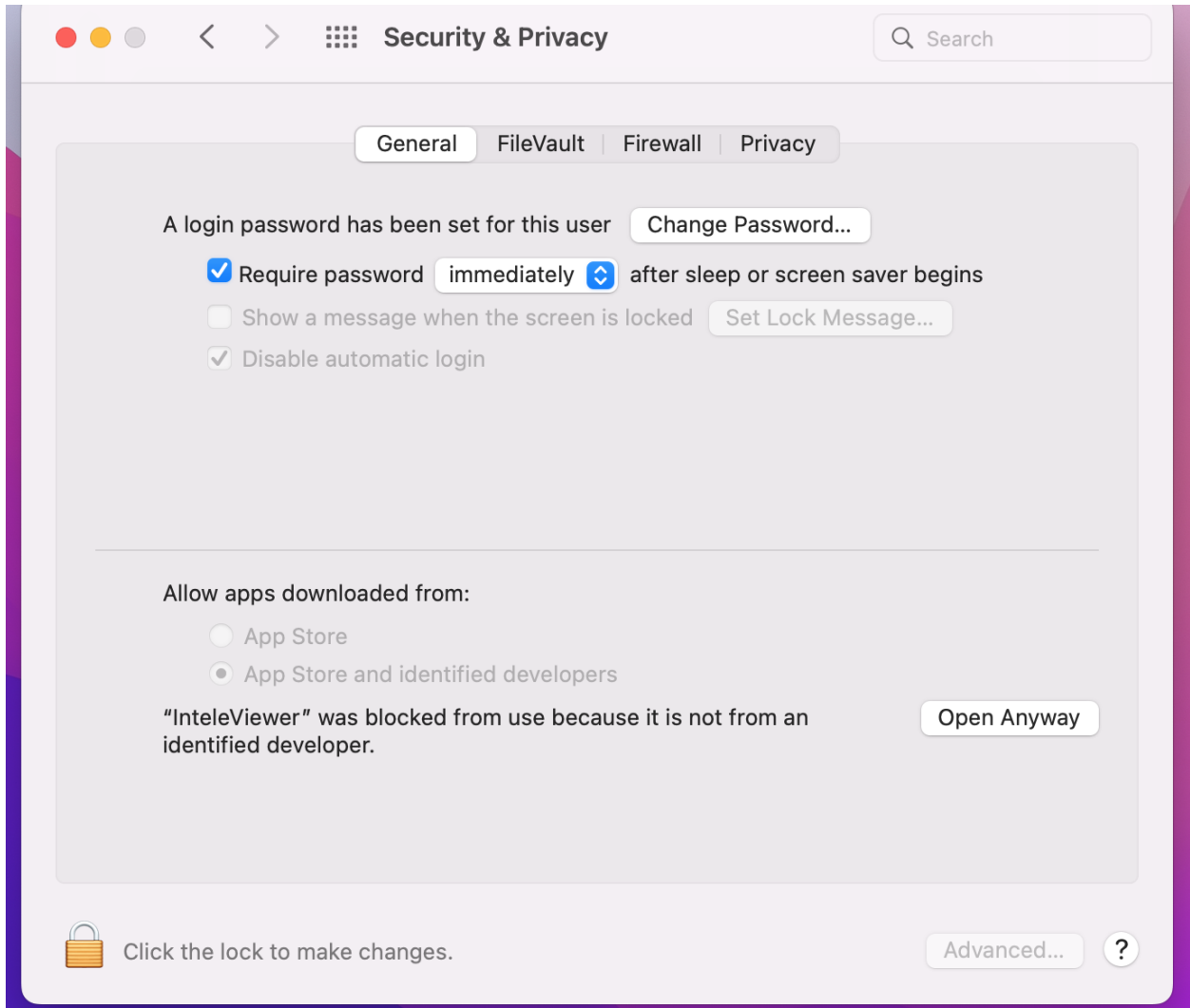

9. Navigate back to Applications and open IntelViewer

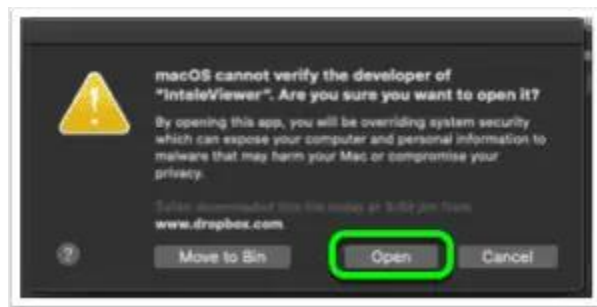

10. If you have not had IntelViewer on this computer before you will need to add the bookmark for San Radiology, click on **EDIT BOOKMARKS**



11. The IntelViewer / PACS bookmark is: <https://pacs.sanradiology.com.au>

You can give it any name you like but we use San Radiology



12. You should then be able to login with your San IntelViewer username and password.

